ACCU-CHEK® Softclix

LANCING DEVICE



Instructions for Use

The ACCU-CHEK Softclix lancing device uses ACCU-CHEK Softclix lancets to obtain a drop of blood from the fingertip. The penetration depth can be selected to match any type of skin.



This cap is for fingertip testing only.



The clear cap is for testing on sites other than the fingertip (AST).

Do not use the clear cap for fingertip testing.

Note: The ACCU-CHEK Softclix cap was not designed to be replaced after each use. The ACCU-CHEK Softclix lancing device is intended for use by a single person and is not suitable for testing different persons with the same device as this may lead to infections.

Selecting the Depth

The depth on the ACCU-CHEK Softclix lancing device may be adjusted by turning the Comfort Dial. The current depth selection is shown in the center of the depth indicator window. You can find the depth indicator window as a semicircular cutout on the ejector.

The ACCU-CHEK Softclix lancing device offers 11 depth settings. The settings are shown in the depth indicator window by the numbers 1 to 5 with additional levels marked with dots to enable



you to precisely adjust to your individual skin type. The higher the number visible in the depth indicator window, the greater the depth setting.

Note: If you are using the ACCU-CHEK Softclix lancing device for the first time, select 1 or 2 to see if the amount of blood obtained is enough to perform a test. If your skin is calloused or thick, you may need to try a higher setting.

- To select the desired depth setting, turn the Comfort Dial so that
 the desired setting is in the center of the depth indicator window.
- Select a new depth setting by turning the Comfort Dial. The cap will snap into a new position, preventing any accidental change of the depth setting.



Performing a Fingerstick

1 Pull off the cap of the lancing device. If there is already a lancet in the device, remove it by sliding the ejector forward. Refer to the section called "Removing the Lancet."



- 2 Firmly insert a new ACCU-CHEK Softclix lancet in the lancet holder until it snaps into place.
- Twist the protective cap on the lancet several times and pull gently to remove it.
- Snap on the cap after lining up the notches. The cap snaps easily into place.





- Select the desired depth setting as instructed in the section "Selecting the Depth." Then, press the plunger on the end of the device all the way down, and you will hear a click. When the device is ready, you will see a yellow dot appear and remain in the center of the clear release button.
- Now, wash your hands and dry them thoroughly. Also warm the fingers to increase blood flow. If you are using alcohol wipes, make sure your finger is completely dry before continuing.
- Grasp your finger near the area to be pricked and gently massage for three seconds.
- 8 Hold the lancing device firmly against the side of your fingertip, then press the release button. Gently squeeze your fingertip to get a small drop of blood. The best place to test on the fingertip is the side, because it has the



best blood supply, yet allows for the least painful fingerstick. Any finger may be used for a fingerstick.

- Apply the drop to the test strip you are using.
- Immediately remove used lancet. To avoid accidental injury, do not store lancet in device.

Removing the Lancet

- To remove the lancet, pull off the cap of the ACCU-CHEK Softclix lancing device.
- 2 Hold the lancing device so the lancet points downward. Slide the ejector forward to release the lancet into a puncture-proof container.



For least painful testing and to avoid infection, use a new lancet each time you perform a fingerstick.

Getting Blood from a Site Other Than Your Fingertip

Testing on sites other than the fingertip is referred to as alternate site testing (AST). These sites may be any of the following – palm, forearm, upper arm, thigh, or calf. We suggest the palm as the nonfingertip test site.

Please refer to your test strip insert or your blood glucose meter's owner's booklet to see if your meter is designed to test blood glucose from sites other than your fingertip as well as for specific AST testing instructions.

IMPORTANT – There is a delay (lag time) in the delivery of blood glucose to nonfingertip sites when compared to fingertip. Therefore, there is a potential for a significant difference between fingertip and AST results. The length of delay (lag time) is strongly impacted by lifestyle, foods eaten, and any other event having impact on blood glucose levels. It is important that people considering nonfingertip testing understand the following:

- Talk to your doctor if you are considering testing from sites other than your fingertip.
- AST testing is not recommended during periods of rapid decreases or increases in blood glucose levels.



05104076001(03) V1/R1 (black) - 2010-05

pi_05104076001_03_En-US_420x216.indd 1

- AST testing is not recommended for people with a history of recurrent hypoglycemia, who suspect their blood glucose is extremely low, or who are unaware of hypoglycemic events.
- NEVER ignore symptoms of high or low blood sugar.
- Do not change your treatment based on one result.
- If the AST reading does not match how you feel, perform a fingertip test to confirm your results. If your blood glucose still does not reflect the way you feel, contact your doctor. For more information on AST testing, please contact the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.
- Wash the nonfingertip site with warm water.
- 2 Make sure that you are using the clear cap specially designed for AST testing with the ACCU-CHEK Softclix lancing device.
- 3 Adjust the clear cap to the highest setting.
- Press and hold the lancing device against a fleshy area on the AST site. Press the release button and continue to apply steady pressure to the site.

Don't be afraid to apply strong pressure to the area with the lancing device to get a good blood drop.

Caring for Your Lancing Device

- Your lancing device requires very little care.
- Keep it away from extreme temperatures.
- Clean the outside of the lancing device regularly with 70% isopropyl (rubbing) alcohol. Let it air-dry.
- Do not dip the lancing device in liquid.
- Do not use bleach to clean the lancing device or caps.

Assistance Available

If you need assistance, please contact the ACCU-CHEK Customer Care Service Center toll-free, 24 hours a day, 7 days a week. We offer assistance in Spanish and many other languages.

1-800-858-8072 phone 1-800-858-8074 TDD/TYY (for the hearing impaired)

Also Available from Roche

ACCU-CHEK Softclix Lancets

Softclix 100/box Softclix 200/box

One (1) Year Warranty

Roche warrants the ACCU-CHEK Softclix lancing device against defects in materials and workmanship for a period of one (1) year from the date of purchase. The liability of Roche under this warranty is limited to the repair or replacement, at Roche's option, of all parts or materials proven to be defective.

This warranty does not cover repair or replacement of ACCU-CHEK Softclix lancing devices that have been subjected to alteration, misuse, tampering, or abuse.

THE FOREGOING WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, WHETHER EXPRESS, IMPLIED, OR STATUTORY. ROCHE DISCLAIMS ANY AND ALL IMPLIED WARRANTIES, INCLUDING THE IMPLIED WARRANTIES OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE. Roche's entire liability with respect to the ACCU-CHEK Softclix lancing device shall be the repair or replacement of nonconforming devices, at Roche's option. In no case shall Roche be liable for incidental, indirect, special, or consequential damages, so the above limitation and exclusion may not apply to you. This warranty gives you specific legal rights, and you may have other rights that vary from state to state.

All inquiries about the ACCU-CHEK Softclix lancing device under the above warranty or service policy must be directed to the ACCU-CHEK Customer Care Service Center by calling 1-800-858-8072. You will be informed of the procedure necessary for replacement. Please DO NOT return any device to Roche without authorization.

IMPORTANT – Read carefully before use. Caution: A restricted license limits the use of this ACCU-CHEK Softclix lancing device.

The ACCU-CHEK Softclix device and its use are protected by U.S. Patent Nos. Re.35,803; 6,419,661; 7,077,828; 7,322,998. A license to use the ACCU-CHEK Softclix lancing device is only granted when ACCU-CHEK Softclix lancets are used with the ACCU-CHEK Softclix lancing device.

ACCU-CHEK Softclix lancets are high precision components that are produced to the close tolerances required for satisfactory operation with the ACCU-CHEK Softclix lancing device. Use of other lancets with

the ACCU-CHEK Softclix lancing device may prevent or impair proper function of the ACCU-CHEK Softclix lancing device.

Using the ACCU-CHEK Softclix lancing device indicates your acceptance of the restricted license to use the ACCU-CHEK Softclix lancing device only with ACCU-CHEK Softclix lancets. Further, if you have purchased an ACCU-CHEK Softclix lancing device that includes this restricted license, then this restricted license applies regardless of any additional offers found in ACCU-CHEK Softclix lancing device packages. If you do not agree to the terms and conditions of the restricted license, you may return, at the place of purchase, the unused ACCU-CHEK Softclix lancing device for a full refund. If you have any questions, please call the ACCU-CHEK Customer Care Service Center at 1-800-858-8072.

Except where prohibited by statute, all warranties covering the ACCU-CHEK Softclix lancing device are voided by use of the ACCU-CHEK Softclix lancing device with any lancets other than ACCU-CHEK Softclix lancets.

Visit our Web site at www.accu-chek.com

This product is covered by the following U.S. Patents: 6,419,661; 7,077,828; 7,322,998; D429,814; 6,464,649; and RE. 35,803.

ACCU-CHEK and SOFTCLIX are trademarks of Roche.



Roche Diagnostics GmbH Sandhofer Strasse 116 68305 Mannheim, Germany www.accu-chek.com

Distributed in the U.S.A. by: Roche 9115 Hague Road, Indianapolis, IN 46256, USA Made in Germany

© 2010 Roche. All rights reserved.



pi_05104076001_03_En-US_420x216.indd_2